### **Automatic Veterinary Syringes**

# ThaMa 216 Automatic Double Syringe

Fix Dosage: 1cc, 1.5cc, 2cc



#### **Operating Instructions:**

This double automatic syringe is precise, reliable and accurate vaccinator, which allows vaccination of two different vaccines at two different dosages at once.

#### **Before Use:**

- It is recommended to sterilize the syringe before each use
- 2. Check that all screwed parts are tightened
- Make sure that the valves, springs and washers are properly positioned into the base.

#### To set the Dose:

- Unscrew the banding screw (26) and pull out the syringe.
- 2. Unscrew the body cover (13) and pull out from the body (01).
- Open the screw nut M4 (32) and unscrew it out
   - you can use the key-pin into the hole of the piston.
- Take the spring (15), body cover (13) and put them on the new piston (11) with the measure you need.
- 5. Replace the cover body, spring on the new piston and screw the nut screw M4 in his position.
- 6. Put the O-Rings on the piston.
- Insert the new pistons with the cover to the cylinder into the body.
- 8. Put the syringes back in their housings (25) Make sure that the top nut is in its hole at the

- pistons shove (21) and screw the banding screw (26) tightly.
- Make sure that you set the dosage in both syringes.

#### How to set the Bridge 2 to 1 Needle:

- 1. Unscrew the needle adaptor (02) from both syringes take care of the springs and valves.
- 2. Insert the springs (05) into the Bridge and the Valve balls (06) in the right position springs in the bridge and balls in the body.
- Close the bridge in to the body's and tight by hand.

#### Care and Maintenance after Use:

At the end of every vaccination day:

- 1. Dismount the needles from the syringe
- 2. Take out the syringes from the syringes housing (unscrew No. 26).

#### Do not open the upper handle for cleaning

- Open the body cover and wash it, open the luer lock needle adapter and tubing valve housingwash them in clear water
- Before using again, check that all the parts are in their place and put a drop of castor oil (supplied) on the O-Ring

Make sure that you don't loss the small parts: vales, spring and valves plastic washer

## ThaMa 216 Automatic Double Syringe

