Automatic Veterinary Syringes

2 Needles Fowl Pox Vaccinator



To fill the vaccine:

- 1. Open the end front cap of the vaccinator with the face up.
- Fill the 4cc of the vaccine direct to the glass barrel.
- 3. Close the end front cap make sure that:
 - the neoprene washer is in place.
 - · the 2 needles are in place.
- 4. Tight the end front cap gently make sure that the needles slide smoothly if not, release back a little the nozzle.

The Vaccinator is ready to use.

Cleaning and maintenance:

- Open the end front cap and make sure that the neoprene washer is in place - wash it in clean water.
- 2. Take out the glass barrel and the special rubber stopper wash them in water.

Make sure that the eyes of the needles are free!

You may sterilize the Vaccinator before the next use.

The Automatic Fowl Pox Vaccinator with 2 Needles is made of the best materials, is oil-proof, non-corrosive and can be sterilize up to 125°C.

2 Needles Fowl Pox Vaccinator

