Automatic Veterinary Syringes

ThaMa 252 Automatic Double Syringe

2 different dosages of various vaccines at once with different pistons 0.5cc, 1.0cc, 2.0cc



Instructions:

The ThaMa 252 is double syringe precious, reliable and accurate vaccinator.

ThaMa 252 is for vaccinate two different dosages of various vaccines at once and operate between the five pistons for each syringe!

Before Use:

- 1. It's recommended to sterilize the syringe before each use.
- 2. Check that all screwed parts are tightened.
- 3. Make sure that the valve springs and washers are properly positioned into the base.

How to set the dose:

- 1. Unscrew the banding screw (#26) and pull out the syringe.
- 2. Unscrew the body cover (#13) and pull out from the body (#01).
- 3. Insert the key-pin (#20) into the hole on the piston.
- 4. Hold the top nut with tool and unscrew it out.
- 5. Take the spring (#15) and the body cover (#13) and put them on the new piston (#11) with the measure you need.
- 6. Replace the cover body, spring and the screw nut M4 (#32) in order.
- 7. Tight the screw nut (#32): insert the key-pin into the hole on the piston and tight gently with tool.
- 8. Put the syringe back in its housing (#25) (make sure that the top nut is in its hole at the piston shove) and screw the banding screw (#26) tightly.

Make sure that you set the dosage in both syringe.

One of the syringes is marked!
That make easier to remember in which syringe you have one of different dosages during the working day.

How to set the bridge 2 to 1 needle (#27):

- 1. Unscrew the needle adapter (#02) from both syringes (take care of the springs and the valves).
- 2. Insert the springs (#05) in the 2 to 1 bridge and the valve in right position (valve in the body and spring in the bridge).
- 3. Close the 2 "bridge covers" upon the syringes and tight them gently.

Care and Maintenance after Use:

- 1. Dismount the needles from the syringes.
- 2. Take out the syringes from the syringe housing (unscrew #26).
- Open the body cover and wash it, open the luer lock needle adapter and the tubing valve housing – wash them in clear water.
- 4. Before using again, verify that all the parts are in their place and put a drop of castor oil (supplied) on the O-ring.

Do not open the upper handle for cleaning!

Sterilization:

All parts of the syringe can be sterilized. It is recommended to sterilize after every use. Up to 125°C in autoclave

ThaMa 252

2 different dosages of various vaccines at once with different pistons 0.5cc, 1.0cc, 2.0cc

252-01	- Syringe	body
--------	-----------	------

252-02 - Luer lock, needle adapter

252-03 - Tubing valve housing

252-05 - Valve spring

252-06 - Valve stainless ball

252-07 - Valve plastic washer

252-10 - Piston o-ring

252-11 - Piston

252-32 - Screw nut M4

252-13 - Body cover

252-15 - Handle spring

252-16 - Finger grip - handle

252-17 - Upper handle

252-21 - Piston shove

252-22 - Shove shaft

252-23 - Spacer

252-20 - Key-pin

252-25 - Syringe housing

252-26 - Banding screw

252-27 - Bridge 2 to 1 needle

252-28 - o-ring gasket for 27

252-71 - Valve fiber washer

252-99 - Spare parts-kit

(include: 252-05*2, 252-06*2, 252-07*2,

252-10*4, 252-12*1, 252-26*1, 252-71*1,

252-28*1)

